REDAX®

# DRENTECH PUMP DRENTECH CLOSED

IT

## SERBATOIO IN SILICONE

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#### DESCRIZIONE

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### INSTRUCTIONS FOR USE

#### DESCRIPTION

The drainage system is composed of a silicone reservoir equipped with an anti-backflow valve and one or two connectors for connection to the drainage tube. The reservoir may have a capacity of 100, 200 and 400 cc and can be equipped with a collection bag. Thanks to the typical elasticity of silicone, the drainage suction power remains constant until the reservoir is completely full.

#### INTENDED USE

The drainage systems are intended to be used for post-operative evacuation of fluids from the surgical or traumatic wound preventing fluid build-up.

**Note:** This device can be used in safe conditions only in the field of application and according to the procedures indicated in this instruction leaflet in correspondence to the type of product. The manufacturer declines all responsibility for improper use or different from that indicated.

#### WARNINGS

Do not use if the package has been opened or damaged. Avoid exposure to high temperatures and ultraviolet rays during storage. Dispose of the device after each single use; do not reuse. Adopt adequate precautions for elimination and disposal of the device and comply with the provisions of the laws in force on biologically hazardous waste.

#### INSTRUCTIONS FOR USE

##### Preparing for use

Figure 1: Connect the free end of the drainage tube (A) to the tapered connector (B) of the silicone reservoir; if using only one drainage, seal the second connector with the dedicated cap provided.

Figure 2: For the versions with bag, first close the clamp on the drainage connector. Holding the silicone reservoir firm in the palm of your hand, start applying moderate pressure until it is completely compressed.

Figure 3: Once the reservoir has been compressed, close the primer valve (C) with the dedicated cap. Releasing the hand that was compressing the reservoir, drainage tube suction will immediately be activated.

Figure 4: Warning: do not compress the silicone reservoir pushing in from the bottom. This would deactivate the suction system.

##### What to do when the reservoir is completely full

When the reservoir is completely full, it must be replaced with a new device. To carry out this operation, clamp the drainage tube and remove the device in use. The anti-backflow valve prevents accidental fluid spillage. Proceed with connecting a new device following the instructions above and then remove the clamp from the drainage tube.

##### Version with bag

In the version with bag, you can transfer the fluid collected, restoring the suction capacity of the reservoir. To do the transfer, open the clamp on the drainage connector and squeeze the reservoir with both hands until it is completely empty (Fig.5). If done properly, this operation allows automatically restoring the suction capacity.

If the collection bag is completely full, you can replace it; close the clamp on the drainage connector and then remove the bag in use and connect a new one. If the bag

EN

is equipped with a tap, you can use it to empty the bag and restore the collection capacity.

#### CONTRAINDICATIONS

There are no known contraindications.

**STERILE** - Sterilized by ethylene oxide. The product is sterile if the package is intact.

**SINGLE-USE DEVICE**. Reuse may lead to alterations in performance and cross-contamination risks.

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