**REDAX®**

**IT**

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## SMART DRAIN-SPIRAL DRAIN™ - KARDIA SPIRAL Round Fluted Drain FLAT FLUTED DRAIN

### INSTRUCTIONS FOR USE

#### DESCRIPTION

The biocompatible pure silicone flat or round drains, SPIRAL DRAIN and SMART DRAIN, are formed by:

- SPIRAL DRAIN:**
- a part of radio-opaque white silicone pipe with grooves.  
This exclusive design allows an efficient drain because of the capillary action and avoids the occlusion of the pipe as a result of tractions and/or torsions.
  - a part of transparent silicone pipe, connected to the radio-opaque end following an atraumatic profile, through which it can be connected to any closed drain systems by gravity or intake.
- SMART DRAIN:**
- a sole length of silicone pipe at the distal end of which the profile has some grooves that, seamless, converge in a round profile part at the proximal end. So, the diameter absolute regularity, which makes this device suitable to be used in laparoscopy and thoracoscopy, is combined with the drainage efficiency of the fluted profile. Transition between the fluted profile and the round profile is quick in order to avoid the development of channels with a little diameter which could be obstructed by clots.

Depth marks, graduated from the bottom of the radio-opaque end, make easy to position and remove the drains. All the drains are available in many dimensions and with steel Trocar needle. All configurations include a sterile double envelope wrapping and contain a special union, in sterile wrapping, for the connection to the collecting devices.

#### INDICATIONS

The drainage is intended for short-term use in clinical conditions that require evacuation of fluids that have accumulated in natural and/or newly-formed cavities following surgery. The SPIRAL DRAIN tube is suitable for use in conjunction with gravity or suction drainage systems.

**Note:** This device can only be used in safety conditions in the field of applications and using the procedures indicated on this operating instruction sheet for the specific type of product. The manufacturer declines all liability relating to improper use or use different to that indicated.

#### SUGGESTIONS FOR USE

Positioning the drainage.

1. Before positioning the drainage, carefully select the type, the position and the size required. These parameters depend on the type of operation, the physiology of the patient and the experience of the physician.
2. Using an aseptic technique, carefully remove the sterile thoracic drainage from its protective wrapping.
3. Position the drainage in the wound using an aseptic procedure. With products fitted with trocar needle, use the latter according to the usual positioning technique, working from the inside to the outside.
4. For proper positioning use the depth markings indicated on every drainage device.
5. Fix the drainage by means of a stitch or plaster.
6. Connect the positioned tube to the selected drainage system following the instructions on the relevant instruction leaflet. In the case of drainages fitted with trocar needle, cut the latter and dispose of it before connecting to the drainage system. If necessary, the connector provided in the package can be used (where provided) to make the connection to the drainage system.
7. It is inadvisable to use drains with high-vacuum bottles.

Removing the drainage.

1. Remove the tape or stitch or any other type of fixing.
2. Disconnect the drainage system.
3. Remove the drainage by pulling slightly and immediately apply a wound dressing.

#### CONTRAINDICATIONS

There are no known contraindications.

#### GENERAL PRECAUTIONS

The product is sterile if the package is intact. Single-use device. Reuse may lead to alteration of performance and risks of cross-contamination. Discharge after each single use, do not re-use. Avoid exposure to high temperatures and ultraviolet rays during storage. For discharging and disposal of the device, adopt suitable precautions and act in accordance with the provisions of the law in force for biologically hazardous waste.

**STERILE - Ethylene oxide gas sterilized.**

#### MATERIALS USED

Silicone biocompatible and hemocompatible.

LATEX-AND PVC-FREE MEDICAL DEVICE.

#### WARNINGS / PRECAUTIONS

1. **Before inserting the drainage, identify the introduction point and select the correct size.**
2. **The choice of drainage size is based on the type of pathology and surgery and the experience of the physician.**
3. **It is absolutely forbidden to modify the drain and make additional holes with any tool. It is allowed only to reduce the length of the fluted part, provided that this operation is executed with a clean cut, without any lacerations, and that the remaining part is 15 cm at least, so the drainage power is not reduced.**
4. **After extraction ensure complete integrity of the drainage, an essential condition to exclude that drainage fragments caused by accidental rupture remain in the cavity.**
5. **It is recommended not to use the drainage tube for a period of more than 29 days.**
6. **After positioning the drainage, ensure that the perforated section is completely inside the wound and that the connections are perfectly sealed.**
7. **The drainage must be removed by hand, pulling slightly and avoiding sudden movements. Avoid the use of metal instruments such as forceps or devices that might cause rupture of the drainage.**
8. **If the drainage is left in the site for very long periods of time, it might be difficult to remove it. Greater attention must therefore be paid during removal.**
9. **Do not use roller clamps or similar metallic milking device on silicone tubes.**
10. **Do not use any abrasive tools, such as cotton flock or gauzes. Milking the silicone tubes by means of those tools can make damages on drain surface reducing the mechanical resistance.**
11. **Do not use any alcoholic or aggressive liquid to lubricate the surface of the tubes before milking. Redax strongly suggests Vaseline oil or cream to help the tube stripping or any other non-aggressive lubricant.**

Date of issue of the last version :  
see last page : (REV.: XX-XXXX)

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